


EYEZEN™+ CUTOUT CHART

Better Sight.
Better Life.



Recommended Minimum Fitting Height 15mm

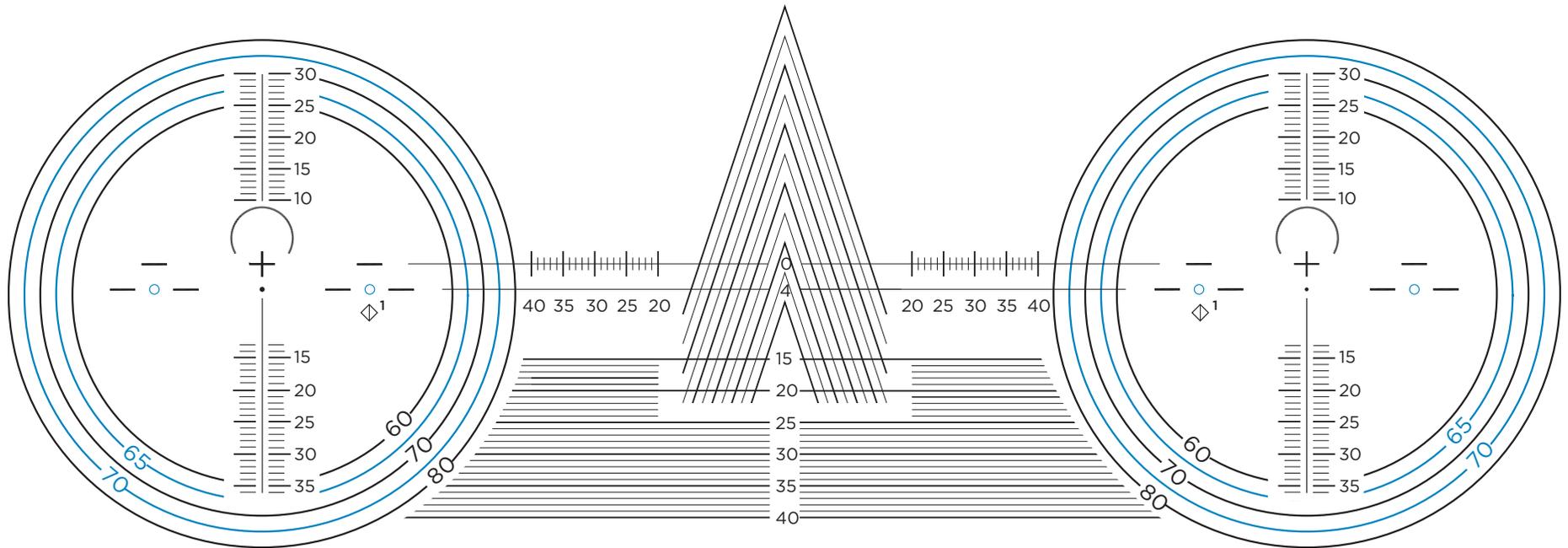

◇¹ Lens Engraving symbol location

LENS ENGRAVING	
PRODUCT	SYMBOL
Eyezen+ 0 (0.00 D)	☺
Eyezen+ 1 (0.40 D)	☹
Eyezen+ 2 (0.60 D)	☹
Eyezen+ 3 (0.85 D)	☹

Eyezen™+

BLOCKING AND FINISHING

Eyezen™ + enhanced single vision lenses are an everyday single vision lens; however, they should be blocked and finished similarly to a progressive lens.

- FRP (Fitting Cross 4mm above the plane of the micro circles) should be used for **blocking for finishing**

Eyezen+ static cling verification decals (LEZN000021) are available from Essilor.

VERIFICATION

Instructions to verify power in Eyezen+ 1, 2, and 3 lenses

- 1 Verify the prescribed power at the distance verification circle
- 2 If prism was prescribed, then verify prism power at PRP
- 3 Confirm engraving to ensure the correct Eyezen+ 1, 2 or 3 was provided*

* Note: No other checks are necessary to confirm accommodative relief

Instructions to verify power in Eyezen+ 0 lenses

- 1 Verify the prescribed power at the fitting cross
- 2 If prism was prescribed, then verify prism power at the fitting cross
- 3 Confirm engraving to ensure the correct Eyezen+ 0 was provided

Patients will not receive the full benefit of the accommodative relief in Eyezen+ 1, 2, or 3 if the Fitting Height is less than 15mm.

Go to [EyezenPro.com](https://www.eyezenpro.com) to find more resources and information on Eyezen+ lenses.

Eyezen™ +