

**Centering:** Center the Sync lens according to the monocular distance PD and make sure that the marking line overlaps the center of the pupil when looking straight in horizontal (zero) distance direction.


**Required ordering data:** Measured prescription for distance refraction and the choice of required vertical support level Sync 5 BKS and Sync 8 BKS.

Compensation of vertex distance and angle of inclination by de-centering (OC in downwards direction) is not necessary.



A=Sync 5    B=Sync 8    R=Right Eye    L=Left Eye

**Materials:**


1=1.50    X=1.53    P=1.59    3=1.60    5=1.67

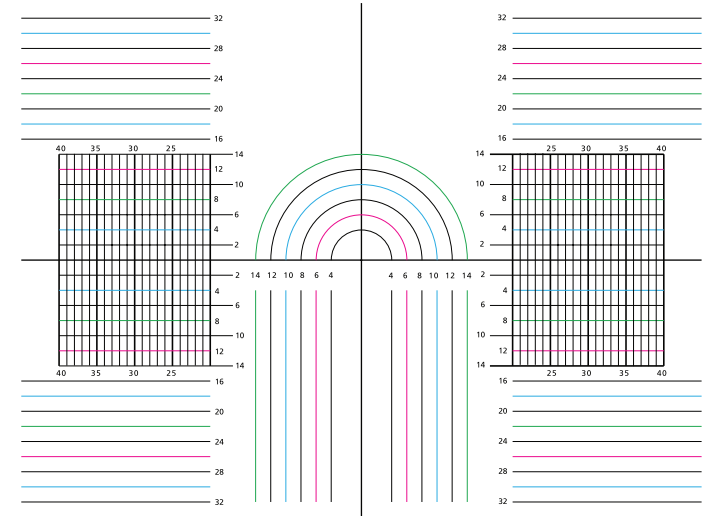

**Sync BKS**

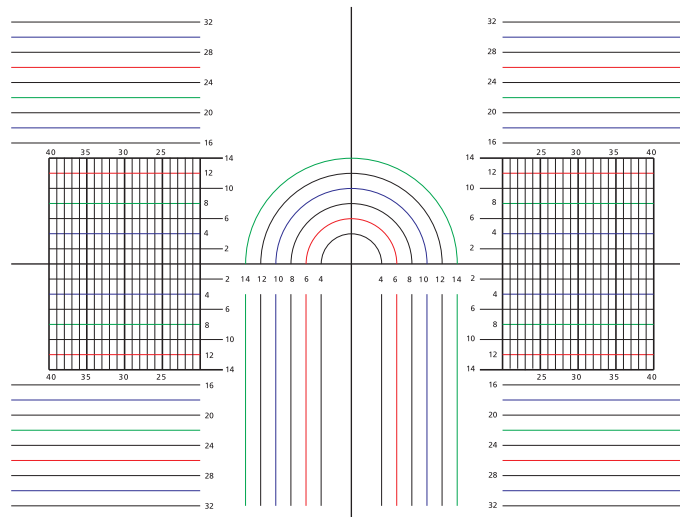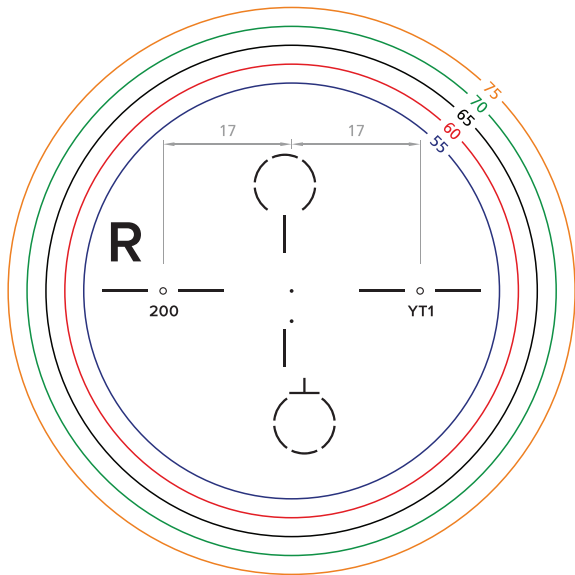

Minimum Fitting Height = 18mm

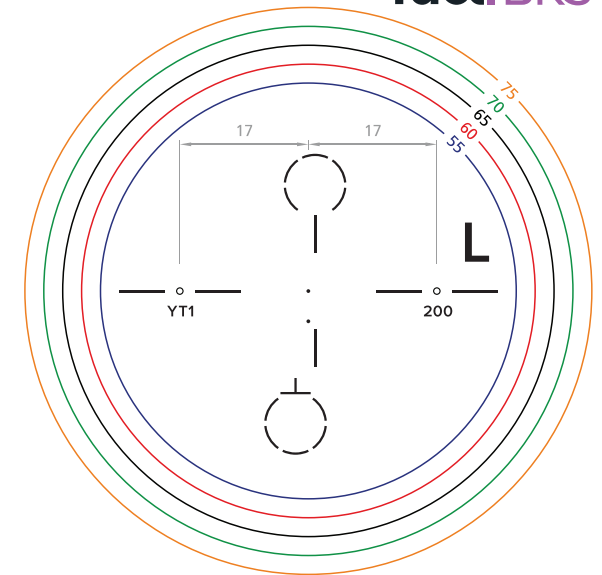

**Tact BKS**

SYNCTCC: 04/17