

KODAK PRECISE PB Progressive Lens Dispensing Aid

KODAK Precise PB Progressive Lenses

1. Select and adjust the frame. Adjust the frame for comfort and accuracy before taking measurements. Adjustable nose pads are recommended. Set the vertex distance 12-14mm. Set the pantoscopic angle to 10-12°. Frame should have a slight face form.



2. PD and Fit Height. Measure monocular fitting heights by marking each demo lens at the pupil centers with a felt tip pen. Measure monocular PDs using a pupilometer or by using the fitting height marks. To translate lens markings into measurements place the frame on the center of the triangle, making sure the marks on the lens are on the zero (0) line. Using the chart, record the monocular PDs and monocular vertical heights.

3. Frame verification. Line up the pupillary mark on the frame with the cross on the chart. **The Blue Optical Zone** is the area required for good distance and near vision. Verify that it is within the eyewire and that the eyewire is within the cut-out diameter. If a small portion of the near or distance zones are outside the frame, the patient may still be satisfied with the frame, as long as the optical compromises are explained.

4. Dispensing. Confirm the monocular PDs and fitting heights. Verify the lens RX on the lensometer. Confirm the fit on the patient by verifying that the fitting cross is properly positioned over the pupils. Adjust the frame as necessary.



5. Teach proper viewing. Demonstrate the different viewing areas, and appropriate head and eye movement.

KODAK Precise PB Lenses
3.0mm Decentered

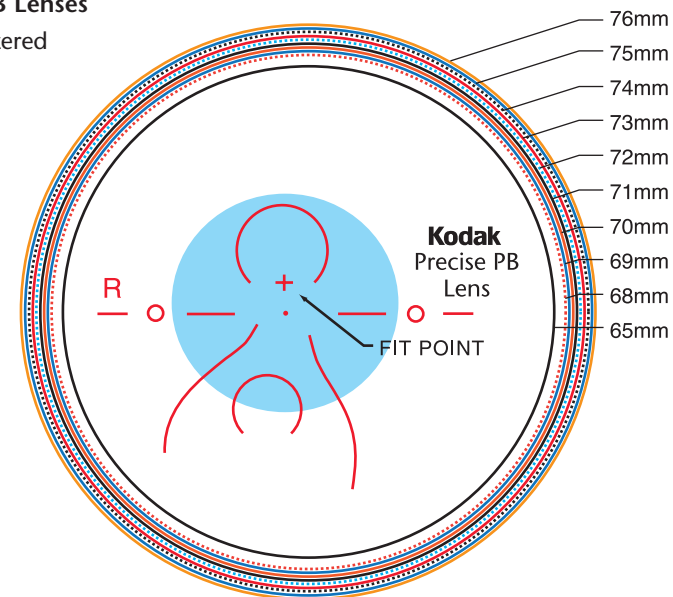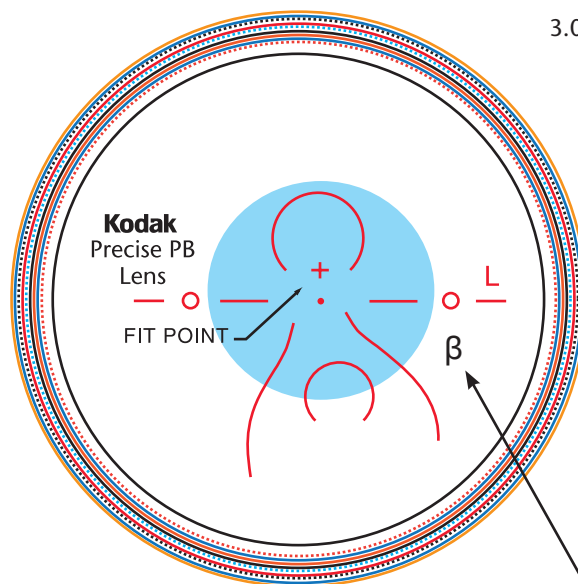

KODAK Precise PB Lens identifying mark

KODAK Precise Short PB Lenses
3.0mm Decentered

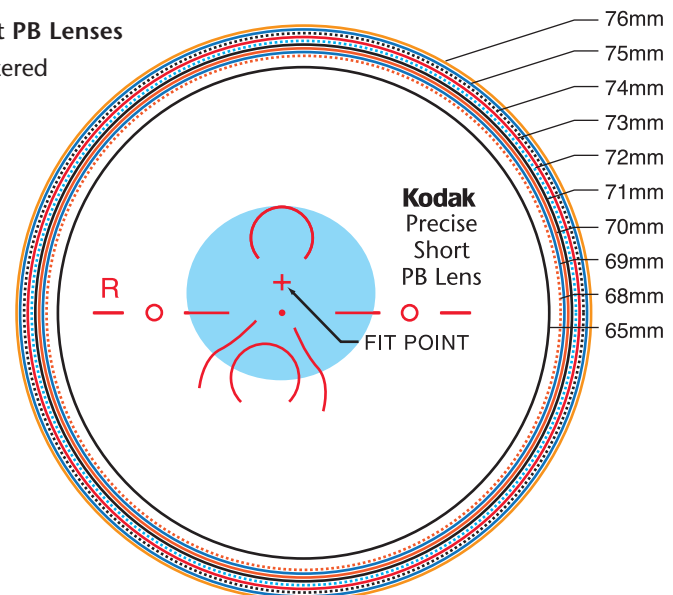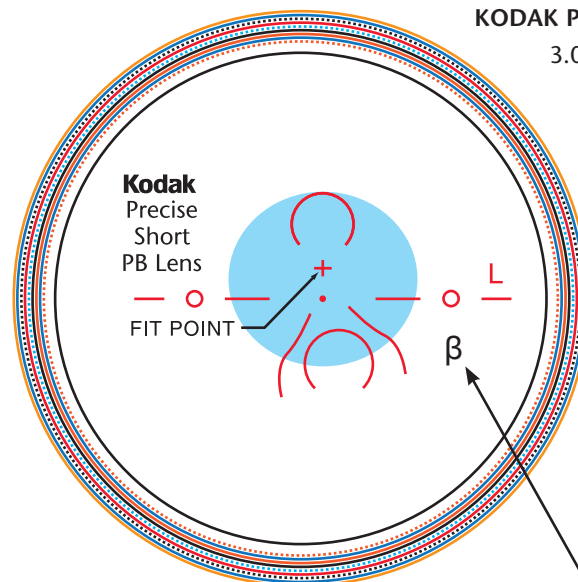


KODAK Precise PB Lens identifying mark

KODAK PRECISE PB Progressive Lens Dispensing Aid



Kodak
Precise PB Lens

Kodak
Precise Short PB Lens

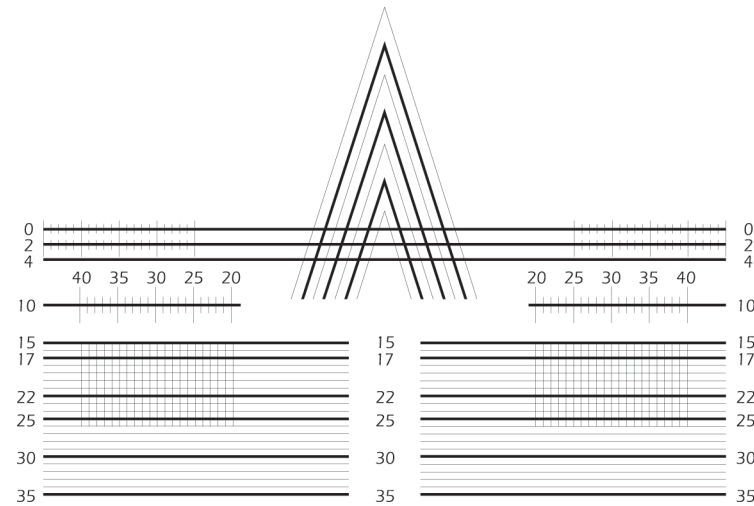

Signet Armorlite, Inc.

5803 Newton Drive, Suite A, Carlsbad, CA 92008 • 800.950.5367 • www.signetarmorlite.com



The Kodak trademark and trade dress are used under license from Kodak by Signet Armorlite, Inc. Precise is a registered trademark of Signet Armorlite, Inc. ©2016 Signet Armorlite, Inc.